

PulsOxy-Fix

Fixation for Pulse Oximeter Sensors

PulsOxy-Fix is a softly padded fixation for pulse oximeter sensors, that is fixed on the skin non-adhesively. It can be applied at the foot, finger or other extremities.



PulsOxy-Fix is available in three sizes:

		Width	Length	SU	REF
1	PulsOxy-Fix 10	1,8 cm	10 cm	120 pcs	30-14
2	PulsOxy-Fix 12	3 cm	12 cm	80 pcs	30-10
3	PulsOxy-Fix 17	4 cm	17 cm	60 pcs	30-12



- ◆ Safe, quick and comfortable handling – even with agitated children
- ◆ Eudermic – softly padded, no sharp edges
- ◆ Non-adhesive – soft to tender skin
- ◆ Latex-free – hypoallergenic
- ◆ Protection against decubitus ulcer
- ◆ Individually adjustable – can be cut to length on request
- ◆ Economic – can be opened and closed at any time



its-novo.com



INSTRUCTION FOR USE

PulsOxy-Fix



1 | Open the sleeve, remove the cover of the adhesive layer and fix the pulse oximeter sensor on it.



2 | Place the pulse oximeter sensor at the foot respectively at the desired extremity.



3 | Wind the sleeve once around the sensor and close the sleeve with the hook and loop fastener.



4 | Fixation in final position.



PulsOxy-Fix 12
Fixation in final position.



PulsOxy-Fix 10
Fixation in final position.



1846 5th Concession W.,
P.O. Box 7, Rockton, ON LOR 1X0
Tel.: 519-622-4030
Toll: 1-800-996-6674
Fax: 519-622-142
mmsi@mcarthurmedical.com
www.mcarthurmedical.com