

# Ambu® Disposable PEEP Valve MR Conditional Valves



## Key Benefits

- Ambu PEEP Valves have been designed to provide unique resistance characteristics when used with manual resuscitators, ventilators, anaesthesia machines and CPAP systems where specified by the manufacturer
- Ambu PEEP Valves can be fitted directly to the patient valve of any original Ambu resuscitator without additional connectors

## Ambu® disposable PEEP valve MR Conditional

Positive End Expiratory Pressure is in general used to maintain adequate oxygenation of the blood. PEEP is more specific recommended in assisted ventilation of preterm infants.

Guidelines 2005\*: "Maintaining a positive end expiratory pressure (PEEP) immediately after birth protects against lung damage. PEEP also improves lung compliance and gas exchange. 311,312" (page S121).

PEEP is also recommended in connection with lung injuries for instance drowning.

Guidelines 2005\*\*\*: "The propensity towards alveolar collapse may require the use of PEEP or other alveolar recruitment manoeuvres to reverse severe hypoxemia. 82" The disposable Ambu PEEP valve 20 valve is MR Conditional according to ASTM F 2503.

The Ambu disposable PEEP valve 20 can be used in an MR environment. This means for example in the MR system room or close to the scanner.

The following conditions must be met:

- Static magnetic field of 3-Tesla or less
- Highest spatial gradient field of 720-Gauss/CM or less

The PEEP valve is not intended to be in direct contact with a patient undergoing a MR procedure.

Ambu Disposable PEEP valve 20 is adjustable between 1.5-20 H<sub>2</sub>O cm. It has a 30 mm connector.

## Ordering specifications

Item no.	Description
199 002 020	Ambu Disposable PEEP Valve 20 (Minimum order is 20 pieces)

## Materials


Product	Material
Transparent plastic parts	Polysulphone
Plastic parts of valve body	Polysulphone with delrin disc and silicone oil
Adjustment cap	Polyamide (nylon)
Sealing in adjustment cap	EPDM rubber
Spring	Stainless Steel (non magnetic)



Disposable PEEP Valve 20

## Size

Dimension	
Length	65 mm, diameter 45 mm
Weight	40 g

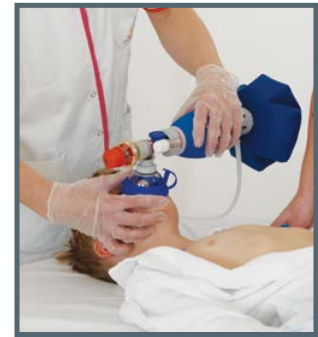

Disposable PEEP Valve 20 MR Conditional mounted on Mark IV Baby resuscitator

**Ambu Inc.**  
 6740 Bay Meadow Drive  
 Glen Burnie, MD 201060  
 Tel. 1 800 262 8462  
 Fax 1 800 262 8673  
 www.ambuUSA.com

**CE** 0086 US: Rx only

900 002 103 - V01 - 07/2012 - Ambu Inc. Technical data may be modified without further notice.

**MMSI**

**McARTHUR**  
 MEDICAL SALES INC.

1846 5th Concession W., P.O. Box 7, Rockton, Ontario LOR 1X0  
 Tel.: (519) 622-4030 • Toll Free: 1-800-996-6674 • Fax: (519) 622 - 1142  
 Email: [mmsi@mcarthurmedical.com](mailto:mmsi@mcarthurmedical.com) • Web: [mcarthurmedical.com](http://mcarthurmedical.com)