




EFFECTIVE AGAINST CORONAVIRUS

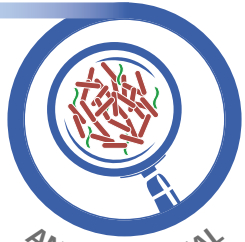

ANTI-MICROBIAL


**ABLE**  
SPACER<sup>®</sup>



**A2A**<sup>®</sup>  
Aerosol to Airways



POSITIVE WHISTLE MASK

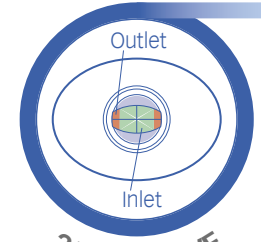

2 WAY VALVE

A recent test conducted in the EU has revealed that **Human Coronavirus, when on the surface of ABLE Spacer and A2A Spacer, is neutralised** within eight hours.

Between a morning dose of inhaled medication and an evening dose the anti-microbial in ABLE and A2A Spacer can largely **neutralise Coronavirus contamination** making it **safe**.

**Click Here**  
for full Press Release

Silver ions in the polymer of ABLE and A2A Spacer are made with anti-microbial properties; the **only known spacer devices on the market to incorporate anti-microbial**.

BPA AND LATEX FREE



**ABLE SPACER<sup>®</sup> & A2A SPACER<sup>®</sup>**

proven to be 97% effective against the Human Coronavirus\*



1846 5th Concession W., P.O. Box 7, Rockton, Ontario LOR 1X0  
Tel.: (519) 622-4030 • Toll Free: 1-800-996-6674 • Fax: (519) 622 - 1142  
Email: [mmsi@mcarthurmedical.com](mailto:mmsi@mcarthurmedical.com) • Web: [mcarthurmedical.com](http://mcarthurmedical.com)



[www.clement-clarke.com](http://www.clement-clarke.com)

\*Data on file (2021)



The A2A Spacer is a pocket size, valved holding chamber (VHC); collapsible for improved portability.

The A2A Spacer is made from anti - microbial, low - static, transparent plastic for improved hygiene. A2A Spacer is for single patient use.

### Product Information

A2A combines portability with a plume shaped chamber of 210ml, to minimise drug loss within the spacer.

A2A has been designed with visible valve movement, this helps the carer to see if the patient is inhaling through the spacer.

A2A overcomes the problem of breathe and press co-ordination, giving the patient more of an opportunity to inhale their medication.

### What is this Product used for?

A2A Spacer is designed to overcome inherent problems with pressurised metered dose inhalers (pMDIs):

- Reducing oropharyngeal drug deposition
- Eliminating some inhale/actuate co-ordination problems

Guidelines advocate use of spacers with children and those having co-ordination difficulties.



### Product Features and Benefits

- Collapsible, pocket size, VHC
- Easy to clean
- Inhibits microbial growth
- Low-static property
- Visible valve movement
- Available with or without mask
- Performance validated
- Accepts all pMDIs available
- 210ml, consistent with optimal performance
- Suitable for both children and adults
- Suitable for local assembly

### Product Suitability



## Product Technical Specifications

Dimensions:	155 x 56 x 68mm (open) 103 x 56 x 68mm (closed)
Weight:	56g
Volume:	210ml
Resistance to flow:	10mm of water @ 30 litres/minute

## Packaging Specifications

A2A in a box | **3607100**

Dimensions:	55 x 65 x 103mm
Weight:	72g

A2A (boxed) with a small mask **3607102**

Dimensions:	70 x 73 x 108mm
Weight:	95g

A2A (boxed) with a medium mask **3607103**

Dimensions:	70 x 73 x 108mm
Weight:	103g

## Part Codes

A2A in a box:	<b>3607100</b>
A2A (boxed) with a small mask:	<b>3607102</b>
A2A (boxed) with a medium mask:	<b>3607103</b>
A2A Masks	
A2A small mask	<b>3607125</b>
A2A medium mask	<b>3607126</b>

All figures detailed are correct at the time of printing.

## Consumables

### A2A Masks

Masks specifically designed for use with A2A. Comfortable for the patient and easy to connect; masks are made from phthalate-free, anti-microbial, low-static PVC.



### Visible Valve Movement


Facilitates training; helps carer to see if patient is inhaling appropriately.



## Distributed by:



1846 5th Concession W., P.O. Box 7, Rockton, Ontario L0R 1X0  
Tel. (519) 622-4030 • Toll Free Line 1-800-996-6674 • Fax (519) 622-1142  
www.mcarthurmedical.com • mmsi@mcarthurmedical.com

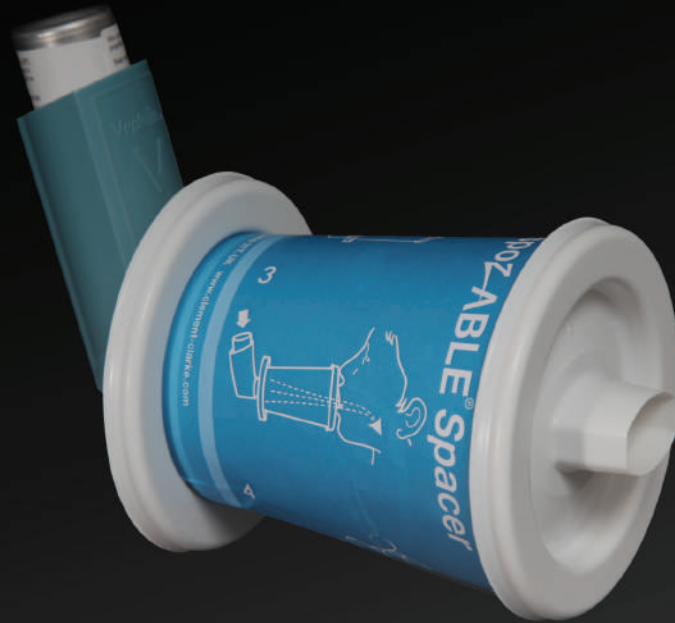

## DispozABLE Spacer

### Technical specifications

# DispozABLE Spacer

## Technical specifications

# DispozABLE™ Spacer

### DispozABLE Spacer

Dimensions	ø81mm x 106mm
Weight	13g
Volume	227ml
Resistance to flow	Free flow

### Performance Data

Fine particle dose (salbutamol):	
Delay 0 secs =	54.5mcg (SD 4.8)
Delay 1 secs =	37.7mcg (SD 7.6)

### Part Codes

DispozABLE Spacer (box of 10)	3609550
-------------------------------	---------

## Product information

DispozABLE Spacer has been designed to meet a specific market need. It is a low cost, fast response spacer for delivering medication promptly. It is designed to be used straight from the box without pre-washing in an emergency situation, for example in an ambulance or school. DispozABLE Spacer is to be used as a single use spacer. This simple intuitive design allows non-professionals to assemble it quickly, clear instructions are included on the spacer, adding to the element of ease.

- Simple to use
- Effective
- For use in an emergency situation; ambulance or school
- Easy to assemble
- No mask required
- Can be used with tidal breathing technique assisted by Flo-Tone.



1846 5th Concession W., P.O. Box 7, Rockton, Ontario L0R 1X0  
 Tel. (519) 622-4030 • Toll Free Line 1-800-996-6674 • Fax (519) 622-1142  
[www.mcarthurmedical.com](http://www.mcarthurmedical.com) • [mmsi@mcarthurmedical.com](mailto:mmsi@mcarthurmedical.com)